<b>de</b>	Gebrauchsanweisung .....	3
<b>en</b>	User's Manual .....	6
<b>fr</b>	Mode d'emploi .....	9
<b>it</b>	Istruzioni per l'uso .....	12
<b>es</b>	Instrucciones de empleo .....	15
<b>pt</b>	Manual de operação .....	18
<b>ru</b>	Руководство по применению .....	21

## Stripp-Quick

QM-1-048F

- de**
1. Am Knopf in Pfeilrichtung ziehen zum Entstauen.
  2. Am Schnurende (ohne Klammer) ziehen zum Stauen.
- en**
1. Pull the knob in the direction indicated by the arrow in order to release the tourniquet.
  2. Pull the ends of the elastic cord (without clip) in order to stop circulation.
- fr**
1. Pour décompresser, tirer le bouton dans le sens de la flèche.
  2. Pour comprimer, tirer sur l'extrémité du ruban (sans agrafe).
- it**
1. Tirare il pulsante in direzione della freccia per sbloccare.
  2. Per bloccare tirare l'estremità (senza graffa).
- es**
1. Para descomprimir, tire del botón en dirección de la flecha.
  2. Estire del extremo de la cuerda (sin la grapa metálica) para comprimir.
- pt**
1. Para alívio da compressão, puxar o botão no sentido da seta.
  2. Para comprimir, puxar pelo extremo do laço (sem gancho).
- ru**
1. Для для возобновления циркуляции крови потяните за фиксатор в указанном направлении
  2. Для для ограничения циркуляции крови потяните за конец жгута (без стопорного кольца).

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## User's Manual Stripp-Quick Tourniquet

Dear Customers, thank you for choosing a KaWe product. Our products are known for their high quality and longevity.

**Please read these instructions thoroughly and carefully before attempting to use this product and heed the care instructions.**

**Utilization:** The Stripp-Quick Tourniquets are only to be used by authorized personnel.

**Purpose:** They are intended as an auxiliary means for the stagnation of blood, the withdrawal of blood or in the case of infusions.

**Operation:** Single and two-hand operation are possible. To use, place the compression loop around the extremity. Next, the ends of the elastic cord can be pulled with either one or two hands until the blood flow to the extremity has been sufficiently stemmed. To release the tourniquet, simply pull the knob.

**Unsuited use/contraindication of the products:** Any use other than that described here is not in accordance with the intended use of the unit. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

**Further information, maintenance, storage:** When used and stored properly, the tourniquet will serve you many years.

**Guarantee:** When used under normal circumstances and with attention to these instructions (User's Manual), we guarantee this product for two years after the date of purchase. Should you need further information or should your instrument require repair, please contact your dealer!

**Cleaning:** In order to avoid dirt accumulation and dust, please always store the tourniquet in its packaging. The unit can be cleaned with a damp, soft and lint-free cloth. To disinfect the tourniquet, a cloth dampened with alcohol can be used.

**Warning:** No special requirements

**Limitation of reuse preparation:** No special requirements. It is recommended that the instruments be reprepared immediately after their use.

### Instructions

**Workstation:** Remove dirt accumulation on surfaces with a disposable cloth.

**Storage and transport:** No special requirements.

**Cleaning preparation:** No special requirements.

**Manual cleaning:** The concentration of surface disinfectants and/or purifying agents is itemized in the manufacturer operating instructions.

**Automatic cleaning:** Cannot be cleaned automatically.

**Disinfection:** The compression loop and the lock are to be disinfected with helipur for example. The concentrations and reaction times given by the manufacturer must be followed.

**Maintenance:** Defective parts must be sorted out and/or replaced.

**Check and performance test:** The smooth running of all movable parts (such as set-screws) must be ensured.

**Packaging:** Individual packaging: a standard polyethylene bag can be used. The size of the bag must correspond to the size of the instrument so that it closes properly.

**Sterilization:** No sterilization required, as it is not a surgical instrument.

**Storage:** No special requirements

**Additional information:** No special requirements

**Common Accessories:** Further information about accessories can be found on our homepage at: [www.kawemed.de](http://www.kawemed.de).

Manufacturer: **KaWe**

**Contacting the manufacturer:** Address or phone number of the dealer or dial the number +49-7141-68188-0.

#### Symbol key:

	Manufacturer		Heed the User's Manual
	Date of manufacture	<b>LOT</b>	Lot code
	Warning!	<b>REF</b>	Product reference number
	Complies with relevant EU guidelines		

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**KIRCHNER & WILHELM** GmbH + Co. KG